Operator's Manual

Model: 5.60-0
Type: Spa Bed

Optional equipment: - Chromotherapy
- "Melody"

INDEX

	page no.
O. Application area	4
1. Delivery	4
2. Apparatus description	5
2.1 System components	5
2.2 Optional equipments	6
3. Indications to risks	7
4. Operation	8
4.1 Control panel view	8
Legend	8
Indications	9
4.2 Operation of the control panel	9
Adjustment of temperature	9
Adjustment of treatment duration	10
Lowering and lifting of the bed	10
Illumination (optional)	10
Melody (optional)	10
4.3 General remarks	10
Mains switch	10
Heating	10
5. Treatment	11
5.1 Consumables	11
Disposable foil 1,6 x 2,1 m	11
Disposable fleece as humidity carrier 1,6 x 2,1 m	11
Towels and bedding aids	11
Cover with fleece inlet	11
5.2 Indications for the treatment	11
Temperature of the bed	11
Protective measures	11
Contraindications	12
5.3 Preparation of the unit	12
Prerequisites	12
Preparation	12
5.4 Treatment with pasty and pulpy media	13
5.5 Treatment with liquid media	13
5.6 Treatment with creamy and oily media	13
5.7 End of treatment	13
5.8 Treatment with insulating underlayer	13
5.9 Treatment media	14

	page no.
6. Cleaning and disinfection	16
6.1 Lying surface (cover)	16
Cleaning	16
6.2 Apparatus cladding	16
Cleaning	16
6.3 Disinfection	16
6.4 Maintenance table	16
7. Maintenance	17
7.1 Cleaning of filter	17
7.2 De-aeration of the water core	17
7.3 Refilling / exchange of water, conditioner	18
7.4 Exchange of the cover	19
7.5 Exchange of the air core	19
7.6 Exchange of the lamp (option)	21
7.7 Maintenance table	22
7.8 Accessories and wearing parts	22
8. Installation and start-up	23
8.1 Assembly	23
8.2 Filling of the device	24
Conditioner	24
Water filling	25
8.3 Electrical connection	26
8.4 Assembly of cladding	27
Assembly of pedestal - U-rail	27
Assembly of lateral parts	28
Assembly of calotte cladding	29
Setting of pedestal strip	30
9. Inspections	31
9.1 Manufacturer's recommendation	31
10. Technical data	31
10.1 Operating conditions	32
10.2 Conditions for transport and storing	32
10.3 Ecological remarks	32
11. Warranty	32
12. Declaration of conformity	33

0. APPLICATION AREA

- The treatment bench is constructed for the application of physiologic, dermatologic and cosmetic health wraps for Wellness and Thalassotherapy.
- **The operation of the unit may only effect under permanent overview of qualified staff!**
- **Helpless and invalid people as well as children may not be treated on the bench!**

1. DELIVERY

- Pack bench pre-installed by factory including:
 - water core
 - air core
 - cover
- Cladding parts:
 - 2 lateral pieces
 - 2 calottes
 - 2 aluminium U-rails
 - 2 aluminium pedestal strips (half-round)
- Accessories:
 - 1 cover with fleece inlet
 - 1 starter kit of foils, 20 pieces
 - 1 starter kit of fleece, 5 pieces
 - 1 conditioner
 - 1 operator's manual
- Installation accessories:
 - 1 filling hose, 10 m
 - 1 funnel
 - 1 bag of replacement rivets, 10 pieces
 - 1 fork for the disassembly of rivets

2. APPARATUS DESCRIPTION

Design and event bench for the application of physiological, dermatological or cosmetic health packs for Wellness and Thalasso.

- Warm and agreeable floating on a water surface (completely free of dents)
- Relaxation by warmth with constant temperature by means of a water bath heating and a cover
- Easy application of health packs by adjustment of the rigidity of the bed

2.1 System components

Chassis and surface of the bed:

- Chassis of coated steel, with adjustable feet for the height adjustment, including design cladding
- Air core for the pneumatic adjustment of the rigidity of the bed, water core for the relaxation on the water's surface
- Integrated heating elements
- The surface of the bed is almost wrinkle-free in rigid state, and therefore easy to clean
- Cover resistant against disinfectants and cleaning agents, grease and oils

Control:

- Integrated micro-processor control with glass control panel
- Operational elements for
 - height adjustment of the bed
 - temperature adjustment
 - adjustment of the therapy duration
- Integrated switch box, splash-water protected IP X4

2.2 Optional equipments

– “Starlight” effect

Spectacular Starlight effects by means of 150 light spots integrated in the bed rim. A total of 8 different colors is available. The colors can be applied changing or in one constant color.

Light system:

Integrated light projector with pale-free color filter and 150 fibre-optics. Power of the light projector: 120 W

Light colors: magenta, salmon, orange, yellow, blue, turquoise, green, transparent

Operation:

Operation to be effected by means of the processor-controlled glass panel.

– “Melody” effect

System for the sensation of music on the whole body, consisting of: complete body music vibration massage by means of transferring the rhythm of the heard music to the water core of the bed.

Music vibration system:

vibration module,

amplifier system incl. connection cable for the connection to the music device delivered by customer.

Operation:

Operation to be effected by means of the processor-controlled glass panel.

3. INDICATIONS TO RISKS

A treatment on this warmth bed by only be effected by an under supervision of trained staff.

Temperatures too high adjusted ($\backslash$ 37 °C) can lead to fever during the treatment course !

⇒ Consider the treatment instructions listed in this manual ⇐

Clients with heart, circulation and blood pressure illnesses may not be treated on this bed !

⇒ Consider the treatment instructions listed in this manual ⇐

Allergic reactions may be a result of incompatibility against certain packs. Therefore we recommend to speak about eventual incompatibilities with the client.

⇒ Consider the treatment instructions listed in this manual ⇐

In spite of a high resistance of the material used, sharp pointed items might damage the bed which will result in a negative influence on the functionality of the device !

Do not shower the surface of the bed for cleaning purposes. Only use humid fabric cloths !

The cleaning of the complete device may only be effected by means of soft cleaning agents. Abrasive or solvent-based cleaning agents as well as strong acids or alkaline solutions are not suitable and may destroy the surface !

The hygienical protection of the water core may only be effected by means of special conditioners for water cores. Other disinfectants can damage the water core !

The bed may not be used in combination with infrared or ceramic radiators, used for the warmth therapy, which are suspended from the ceiling !


The electrical connection of the device has generally to be effected by customer via a mains switch installed in proximity of the device !

In case the device is not filled with water, is has to be cut from the electric supply. The heating operation without water can damage the device !

⇒ Consider the treatment instructions listed in this manual ⇐

4. OPERATION



4.1 Control panel view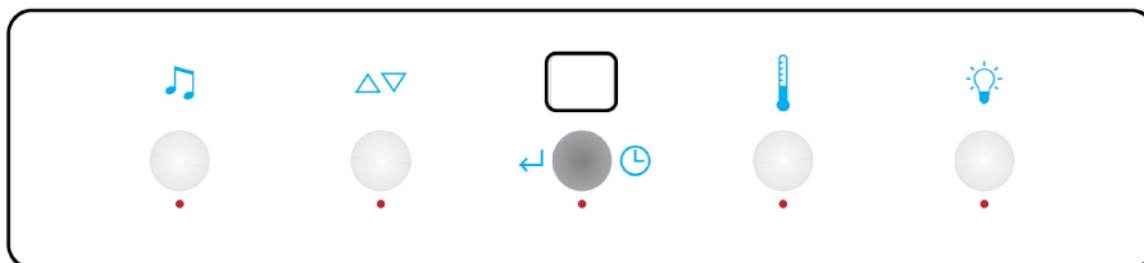

Legend:

Symbol	Meaning / Function / Operation
	Check or adjust nominal temperature Touch key = Indication of the nominal temperature Keep key pushed = Adjust nominal temperature and memorize with "Enter"
	Adjust treatment duration Keep key pushed = Select therapy duration
	Enter Touch key = Selected function is switched on / off
	Lowering / lifting of the bed Touch key and confirm with "Enter" = Lifting / Lowering
	Illumination on / off Touch key and confirm with "Enter" = Lights on / off Select color of lights Touch key = Fix the color
	Melody function on / off Touch key and confirm with "Enter" = Melody on / off

Indications:

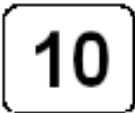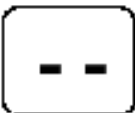Indication	Meaning / Function
	Nominal temperature indication during adjustment or at control of the nominal temperature
	Temperature indication
	No therapy duration adjusted (Manual operation of lifting/lowering function)
 LED flashing	Function selected
 LED lightning	Function activated by "Enter" For the temperature key the lightning LED means that the adjusted nominal temperature is gained

4.2 Operation of the control panel**Adjustment of temperature**

- By touching the "Temperature" key the adjusted nominal temperature is indicated shortly
- By holding "Temperature" key the nominal temperature can be modified within the limit of 25° to 40° C. By a direct confirmation by means of the "Enter" key the selected temperature is accepted as nominal temperature. 32°C to 36°C are recommended temperatures.
- The lightning diode below the "Temperature" key is permanently lightning as soon as the adjusted temperature is reached.

Adjustment of treatment duration

- By holding the “Time-enter” key the therapy duration can be adjusted in steps of 5 minutes and within the limit of 0 - 40 minutes. After releasing the key the adjusted time is accepted and permanently shown in the display.
- In case there is no time adjusted, the display shows 2 bars, which means: “manual operation of lifting and lowering functions”.

Lowering and lifting of the bed

- Touch “Lifting / lowering” key and confirm with “Enter”. The bed is lowered and the therapy course is running. At the end of the treatment the bed is automatically lifted.
- In case there is no time adjusted, there will be no automatic lifting of the bed. By touching key “Lifting / lowering” and confirmation by “Enter” the bed can be manually lifted.

Illumination (optional)

In case the device has been equipped with the illumination function, following operations are available:

- Illumination on / off
- Selection of treatment color
- Touch key “Illumination” and confirm with “Enter”. The rim illumination is lightning, the colors are changing.
- Touch the “Illumination” key once again, and the actually lightning color is permanently fixed.

Melody (optional)

In case the device has been equipped with the Melody function, this function can be switched on and off by single touch of the “Melody” key and confirmation with “Enter”.

4.3 General remarks

Mains switch

The device can be switch on or cut from the electric supply by means of the mains switch installed by customer in proximity of the device.

Heating

When the device is connected to the electric supply (mains switch on), the heating is operational and keeps the bed constantly warm at the selected temperature. In order to avoid temperature fluctuation, the device should remain connected with the electric supply. The heating up of a cold device is approx. 12 hours.

5. TREATMENT

5.1 Consumables

The quantity of the different consumables is depending on the use of pack media.

Disposable foil 1,6 x 2,1 m

Generally for all packs a thin foil is necessary as a layer between the lying surface and the client, in order to protect the unit from impurification caused by the pack mass on the one hand, and in order to protect the pack from getting dry on the other hand.

Disposable fleece as humidity carrier 1,6 x 2,1 m

The humidity carrier is used when the pack mass is too liquid to put it directly on the skin, for example when using whey, herbal brew, milk-oil, brine, hay brew. Here the fleece is soaked in the pack media and put on the client's skin.

Towels and bedding aids

Per client approx. 3 towels are needed:

- A folded towel in order to lay the head on the bed. Alternatively you can take a cushion from our range of accessories
- A towel for the floor
- a bathing towel (large size) or alternatively a bathrobe

Cover with fleece inlet

An impurification-resistant and washable cover is delivered as standard accessory

5.2 Indications for the treatment

Temperature of the bed

The temperature of the bed shall be adjusted between 32° and 36°C. An overheating temperature (>37°C) may only be effected under the supervision of a medical doctor.

Protective measures

Generally protect the client's head from the warmth. Therefore always lay the client's head on an insulating cushion or towel.

Assist the client to get on and off the bed. Caused by the used pack media the surface can get extremely slippery. Therefore also generally put a towel on the floor.

Contraindications

Clients with heart, circulation and blood pressure illnesses may not be treated on this bed !

Allergic reactions may be a result of incompatibility against certain packs. Therefore we recommend to speak about eventual incompatibilities with the client.

5.3 Preparation of the unit


Prerequisites

- The unit is put into operation and the treatment temperature should have been reached
- The surface is blown up wrinkle-free and rigid
If this is not the case, shortly lower the bed and lift it again, now the bed should be rigid
- The device is in hygienically impeccable state

Preparation

- Put the foil in the middle of the bed. Let the lateral parts of the foil hang down
- **Prepare a towel or a cushion for the client's head and for the insulation of heat**

Image 17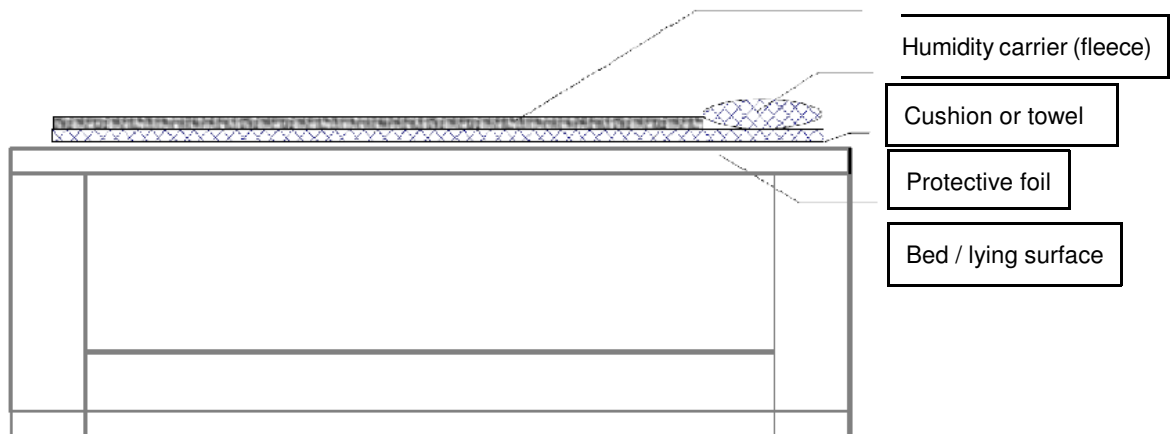

5.4 Treatment with pasty and pulpy media

Is the surface protected by a foil?

- Place the client on the surface of the bed
- Take care for an agreeable positioning
- Put the pasty and pulpy media on the required skin surfaces
- Put the lateral sides of the foil on the client
- Preselect the therapy duration and lower the bed
- Place the cover on the client

5.5 Treatment with liquid media

- Use a fleece or linen as humidity carrier for liquid pack media. Soak it into the media so that it does not drip. Then place the fleece on the foil which you already placed on the surface of the bed
- Place the client on the bed
- Take care for an agreeable positioning
- Put the lateral sides of fleece and foil on the client
- Preselect the therapy duration and lower the bed
- Place the cover on the client

5.6 Treatment with creamy and oily media

- Proceed as described under “pasty and pulpy media”, but use an additional fleece.
- Before starting, check the material compatibility of foil and fleece with the creamy or oil-containing media

5.7 End of treatment

- At the end of the treatment the bed is lifted automatically so that the client can be unpacked carefully.
- Then the customer will go under the shower for cleaning
- The dirty foils and fleeces are thrown away and the bed is cleaned

5.8 Treatment with insulating underlayer

- In case of clients who are sensible against heat, a warmth insulating layer can be put between surface and foil. Suitable are thick, double-woven wool or fleece covers.

5.9 Treatment media

Following table figures traditionally known and actual pack media:

Medium	Indication	Remarks / Treatment durations	Requirement per person
Pasty Media			
Natural mud Natural mud extract Natural peat Fango Salt mud	Rheumatic illness Increase of metabolism Cosmetic application	15 - 20 minutes	approx. 2-3 liters (2-5 kg) 1 foil
Thick wheat bran Wheat bran extract	Eases itching of the skin Cosmetic application	15 minutes	approx. 2-3 liters (2-3 kg) 1 foil
Thick algae Algae extract	Removing slugs Cosmetic application	15-25 minutes	approx. 2-3 liters (2-3 kg) 1 foil, 1 fleece
Liquid Media			
Herbal brew Herbal extract Camomile blossom brew or extract Hay flower brew or extract Spruce needle brew or extract	Removing slugs As herbal description Eases inflammation Cosmetic application Rheumatic illness Neuralgia Cosmetic application Increases blood circulation of skin Relaxing and calming Cosmetic application	Soaked fleece 15-20 minutes	approx. 0,5 liters 1 foil, 1 fleece
Rosemary leaf extract Brine Whey Fat milk White and red wine	Increases blood circulation of skin Cosmetic application Skin diseases, skin care Cosmetic application Rheumatic illness Cosmetic application Eases itching skin Cosmetic application Skin care Wellness Cosmetic application Skin care	Soaked fleece 15-20 minutes Attention: red wine makes stains	approx. 0,5 liters 1 foil, 1 fleece

Medium	Indication	Remarks / Treatment durations	Requirement per person
Creamy and oily Media			
Butter (various)	Skin care Cosmetic applications	Cream in and mass the body softly 15-20 minutes	approx. 50-100 g 1 foil, 1 fleece
Vegetable oil (various)			approx. 20-40 ml 1 foil, 1 fleece
Moisture cream (various)			approx. 50 ml 1 foil, 1 fleece
Body lotion (various)			approx. 50 ml 1 foil, 1 fleece
Greasy cream (various)			approx. 50 ml 1 foil, 1 fleece
Chocolate cream			approx. 50 ml 1 foil, 1 fleece
No additives			
No additional media	Relaxation Pure Wellness	15-20 minutes	Foil or fleece for hygienical reasons

6. CLEANING AND DISINFECTION

6.1 Lying surface (cover)

Cleaning

- Please also see chapter “Indications to risks”
- After the treatment, fold foil and fleece carefully in a way that there will be no pack mass on the lying surface of the bed.
- Give both fleece and foil into the normal domestic refuse
- The cleaning of the device can be effected by commercial cleaning agents. Abrasive or solvent-based cleaning agents as well as strong acids or alkaline solutions are not suitable and may destroy the surface !
- **Do not shower the surface of the bed for cleaning purposes. Only use humid fabric cloths !**

6.2 Apparatus cladding

Cleaning

- Please also see chapter “Indications to risks”
- The cleaning of the device can be effected by commercial cleaning agents. Abrasive or solvent-based cleaning agents as well as strong acids or alkaline solutions are not suitable and may destroy the surface !

6.3 Disinfection

- In case a disinfection becomes necessary for hygienical reasons, you can use commercial, mild surface disinfectants.
- **The disinfectant may not be aggressive against plastics and varnish; special suitability for: PE, PP, PVC.**

6.4 Maintenance table

Kind of maintenance	Intervals	Agents
Lying surface cleaning	after each use	mild cleaning agents (rinsing agents etc.)
Outer cladding cleaning	when necessary	mild cleaning agents (rinsing agents etc.)
Disinfection	when necessary	plastic compatible disinfectants
Polish of cladding	when necessary	commercial polish for varnish


7. MAINTENANCE

Various maintenance works or repairs have to be effected in regular intervals. These are described in this chapter.

7.1 Cleaning of filter

The integrated air turbine is equipped with an air filter which has to be cleaned every 2 years as follows:

1. Remove right lateral cladding (see chapter 8.4)
2. Open the air filter box by means of the butterfly bolt and remove the air filter

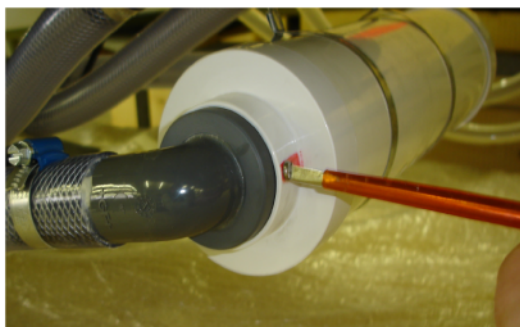3. Clean the air filter and reassemble it
4. Place the cladding

7.2 De-aeration of the water core

When there is a noise of water inside the device, there are too many air bubbles inside the water core. The water core can be de-aerated as follows:

1. Remove calotte cladding below the control panel (see chapter 8.4)
2. Lower the bed and put the device from the current
3. Remove carefully the fixing rivets (11 pieces) of the cover in the front area of the unit by using the fork for the disassembly of rivets, so that the cover can be pulled out with upwards movements

4. Also turn down the air core in order to have access to the water core. The de-aeration valve becomes visible.

Image 19

5. Take out the valve by means of the two flaps, screw off the cover and remove the inner plug.
6. Move the valve gently upwards and go over the lying surface with your hand towards the valve, in order to push the air bubble in destination of the valve.
7. Shut the valve again with the plug and screw on the cover. Important is to set the plug, otherwise the cover does not stay on the thread. The device would get leaky ! Put the valve back into the air cushion.
8. The water level in the water core should correspond approx. to above image. If not, you can refill it on this occasion (see 7.3).
9. Put the air core back into its correct position and fix the function cover by means of the rivets. Press in the rivets completely.
10. Replace the outer cladding and switch on the device.

7.3 Refilling / exchange of water, conditioner

Once a year the water level should be checked, since a low water level can diffuse through the foil of the water core. If the water level is falls, it can result in impairing of the function.

- Filling level (see chapter 8.2 and 7.2)
- Remove calotte cladding below the control panel (see chapter 8.4) and refill the device (see chapter 8.2)
- A refill of the conditioner has to be effected every 2 years only

7.4 Exchange of the cover

Depending to the use and the maintenance of the device, it can become necessary to change the cover of the lying surface (function cover). Fissures, thin spots, etc., are in any way good reasons for changing the cover. Proceed as follows:

1. Remove the outer cladding completely (see chapter 8.4)
2. Lower the bed and cut the unit from the current
3. Remove carefully the fixing rivets of the cover in the front area of the unit by using the fork for the disassembly of rivets, so that the cover can be pulled out with upwards movements

Image 20

4. Insert a new cover and centralize it by means of the distances of the holes
5. Fix the cover by means with the new delivered rivets and press them in completely
6. Replace the outer cladding and switch on the device.

7.5 Exchange of the air core

Depending to the use of the device, the air core can wear out. If it becomes leaky, it has to be exchanged as follows:


1. Remove the function cover as described under 7.4
2. Remove the air hoses of the distributor carefully from the nozzles. Do not damage the nozzles.

Image 21


3. Push the hoses carefully out of the guidance and remove the air core.

Image 22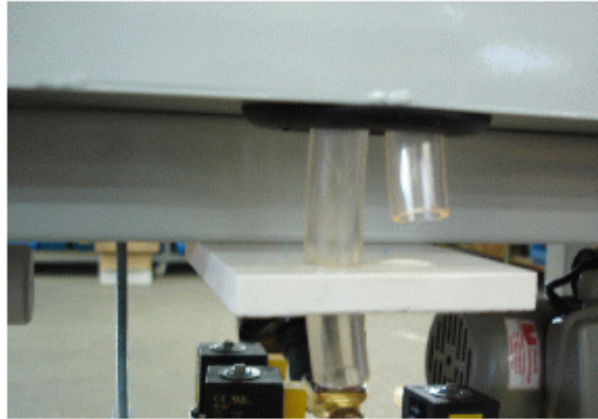

4. Put in the new air core and push the hoses back through the guidance. The ends of the hoses must be on the same level as the tub rim.

Image 23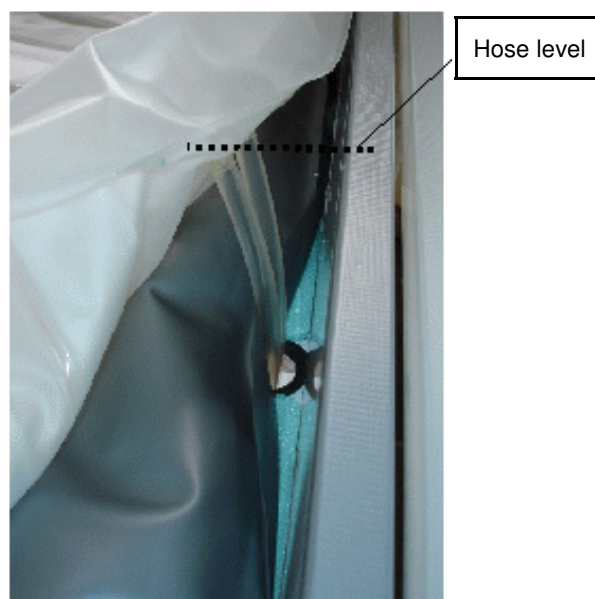

5. Push the air hoses back on the nozzles of the distributor and fix them by means of cable fixing items.
6. Insert the function cover and fix it (see chapter 7.4)
7. Replace the outer cladding and switch on the device

7.6 Exchange of lamp (option)


If the device has been equipped with the optional illumination system, a change of the bulb on the light projector can become necessary, according to wear and tear.

First check that the tub has been cut from the network.

For the replacement of the lamp we recommend to charge an electrician. Do not use others than the a.m. type of lamp. Stronger lamps or those of another type may not be assembled for safety reasons.

Type of halogen lamp: Halogen Photo Optic Lamp 100W/12V/GZ 6,35/24°-25°

1. Remove calotte cladding below the control panel (see chapter 8.4)
2. There is a service cover at the light projector which can be opened by means of a coin to get to the illuminant.

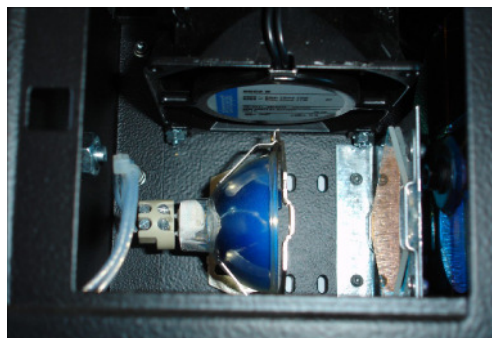3. Withdraw the illuminant from the holder. By means of an appropriate screw driver, the illuminant is removed from the holder.

4. Replace the illuminant.

Please do not touch the bulb or the mirror with your hands, because even a small quantity of grease can make the mirror blind or mat which results in losses of the brightness of the colours.

Projector ventilator

The ventilator grid and the ventilator integrated in the projector should be freed from dust or fluff exchange. (vacuum cleaner) with each illuminant

7.7 Maintenance table

Item	Activity	Interval
Cover of the lying surface	look for fissures, wear and tear	after each treatment
Cover of the lying surface	exchange	when necessary
Lying surface	water level	every ½ to 1 year
Water core	refill conditioner	every 2 years
Air turbine	filter cleaning and/or exchange	every 2 years
Halogen lamp / projector	exchange	when necessary
Projector	eliminate dust	at exchange of lamp

7.8 Accessories and wearing parts

Article	Reference No.
Foil cuts of PE, white, 1,6 x 2,1 m	9-PS-06
Fleece cuts, white, 1,6 x 2,1 m	9-PS-07
Replacement cover	1-5594
Inlet for cover	9-PS-05
Head cushion	1-5595
Cover for lying surface, white	1-5593
Rivets, Nylon	S-NI-64-4
Fork for the disassembly of rivets	9-WZ-001
Conditioner, 118 ml	B-KO-01
Funnel	9-BE-75-01
Water core	9-PS-03
Air core	9-PS-02
Solenoid valve 3/4"	5-MV-20-3/4-01
Control panel, glass	2-ST-07
Control electronic including line	5-NT-03
Air turbine	5-LT-01
Air filter	5-LT-02
Replacement lamp for light projector	5-GL-99-H100

8. INSTALLATION AND START-UP

8.1 Assembly


Do not remove the transport packing before the device is not forwarded to the installation place.

The device must be assembled horizontally. Please use a spirit level. The devices legs are height-adjustable (see images 1 and 2)

8.2 Filling of the device

Connect the delivered hose with the filler cap of the device.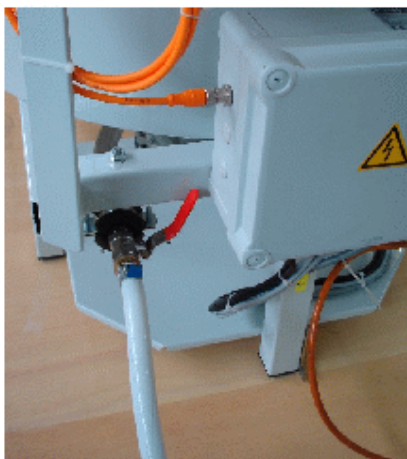

Conditioner


The water filling must be hygienically protected. Therefore use the delivered “conditioner”, which will be added to the filling water.

Use the delivered funnel and fill approx. 100 ml of conditioner into the other end of the filling hose.

Filling with water

- After you added the conditioner, connect the hose with a suitable water tap.
- It is recommendable to fill the unit with tempered water, which will avoid a long heating time.
- First open the valve (image 3) on the suction sleeve of the unit (lever in direction hose = open). Then open slowly the water tap in order to fill the device.

Filling level: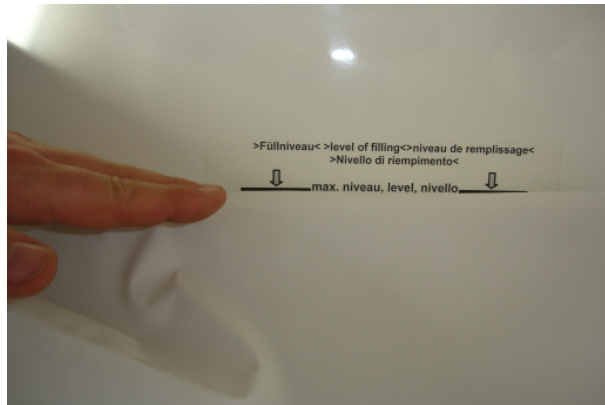

Attention: **Permanently supervise the filling process. An overfilling of the water core can cause damages resp. the bursting of the water core.**


- After the filling, first shut the water tap and then the valve on the suction sleeve (see image 3) (lever direction diagonal to the hose = shut).
- Remove the filling hose from the nozzle.

8.3 Electrical connection


The electrical connection may only be effected by a professional and by means of the delivered installation drawing.

The image shows the connection box for connections: L / N / PE

For more details see chapter "Technical data"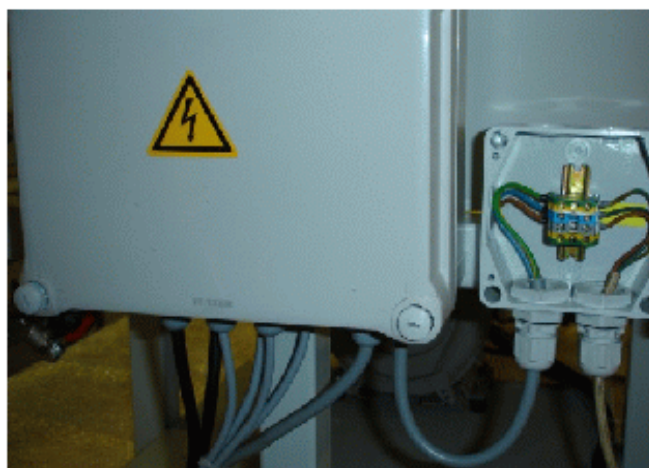

Electrical connection box

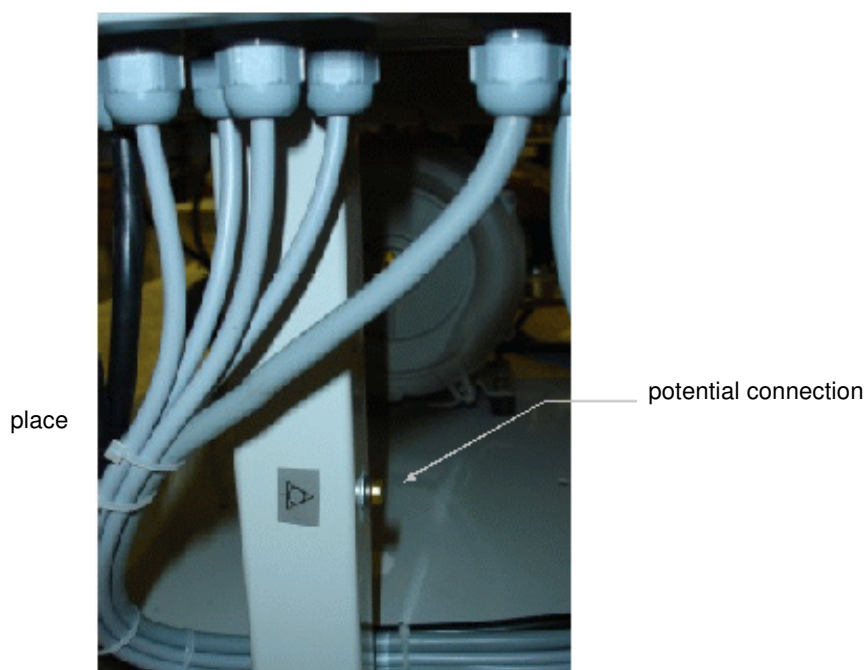8.4 Assembly of the cladding

Assembly and disassembly of the lateral cladding

By unscrewing the two socket screws (4 mm wrench), the lateral cladding is loosened. The socket screws are underneath the offset (see arrow).

Image 8

Image 9

Disassembly of the front cladding

The front cladding parts are loosened from the frame by unscrewing the 4 socket screws (4 mm wrench).

Image 10

After having loosened the cladding on both sides with 2 screws each, lift it so that the socket cladding can be extracted.

Image 11

Setting of the front cladding and the socket

Place the front cladding under the unit and insert it underneath the rim of the unit so that the socket cladding can be insinuated. Fix the front cladding by means of the 4 socket screws (4 mm wrench) not before the socket cladding is correctly placed.

Image 12

Image 13

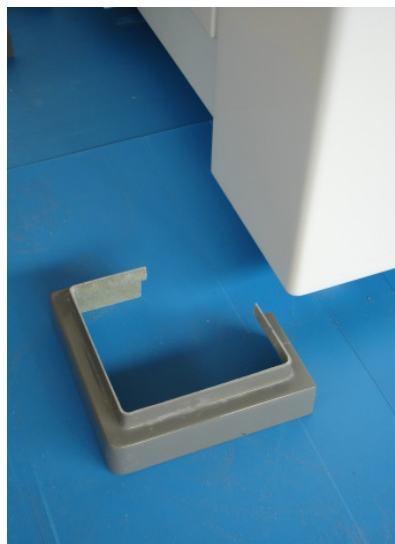
Image 14

Image 15

9. INSPECTIONS

The device described in this manual is subject to the directives of the European low-tension instruction 73/23/EEG and resulting national apparatus safety law. The device must be submitted to a periodical, safety-technical test according to local national accident prevention rules.

9.1 Manufacturer's recommendation

In order to guarantee a safe using of this device, it shall not be set into operation before

- the responsible personal has been schooled by means of the operator's manual in using the apparatus.
- after reparations or technical interventions the device has been submitted to a safety-technical control corresponding to DIN VDE 0751 part 1.
- the device was subject to a 2-years inspection according to the accident prevention rules of DIN VDE 0701 part 1.

The safety-technical checks shall include:

a general sight test	
the protective conductor test	$R = < 0,3 \text{ Ohm}$
the device derivation current test	$I_{EA} = < 7 \text{ mA}$
	admissible for heating operation
the insulation resistance test	$R_{ISO} = > 1 \text{ M Ohm}$
electrical apparatus connection	
function test of the locally required earth leakage circuit breaker (30 mA)	
general control of sources of danger	

10. TECHNICAL DATA

Electrical connection	230 V~, N, PE
Frequency	50/60 Hz
Max. power consumption	1 kW
Protection class	I
Protection category concerning installed components: control box, magnetic valves, air turbine, control panel	X4 IP
Kind of protection	IP X 5
Dimensions (length x width x height)	approx. 2000 x 900 x 700 mm
Interface / conn. to a source of music, entry sensibility	Chinch 110 mV

10.1 Operating conditions

Temperature range	+ 10°C up to +35°C
Relative air humidity non-condensing	up to 90%
Water filling	potable water
Water conditioner	filled up every 2 years, suitable for PVC
Electrical supply - via all poles main switch - power fuse - FI protection switch - min. lead diameter	10 A, resilient, near the unit 10 A, 30 mA, tripping characteristic 1,5 mm ²

10.2 Conditions for transport and storing

Temperature range water core empty	-20°C up to +60°C
Relative air humidity	up to 90%
Transport	Device standing upright

10.3 Ecological remarks

The devices produced by Unbescheiden GmbH, Baden-Baden, are made for the most part of high-quality and reusable materials and should be given after their lifespan to an appropriate recycler.

11. WARRANTY

As manufacturer and supplier of our products we can only guarantee safety and functioning at installation, extension, new purchasing, modification or renovation or repairs, on condition that:

- a) these works have been effected by ourselves or by authorized personal
- b) electrical and sanitary installation of the apparatus and the respective room are corresponding to the appropriate agreements
- c) the apparatus is used by schooled professional personal

Covers, inserts, air core and water core are direct wear and tear parts and are excluded therefore from the guarantee.

12. DECLARATION OF CONFORMITY

Manufacturer / Distributor: Unbescheiden GmbH
Ruhrstrasse 6
D-76532 Baden-Baden
Phone: +49-7221-9534-0
Fax: +49-7221-67186
Email: info@unbescheiden.com

Kind of product: Low-voltage product

Designation of product: Cosmetic treatment bed

Designation of type: Model 5.60-0 "Jouvence"

The described product fulfills the directions of rules:

Low-voltage direction	2006/95/EG
Electromagnetic compatibility	89/336/EG

The device carries the CE mark

Baden-Baden, October 30, 2008

Quality Management*

* Declaration of Conformity is valid without personal signature

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Fabricant / distributeur: Unbescheiden GmbH